Medicop[®]
EQUIPMENT SPECIALIST

Medicop d.o.o.
Nemčavci 81
9000 Murska Sobota
Slovenija

tel: ++386 (0)2 539 12 50
fax: ++386 (0)2 539 12 55

www.medicop.eu

IM.PRB.MAY 2020/V


Medicop[®]
EQUIPMENT SPECIALIST

CE0123

1300004

USER'S MANUAL

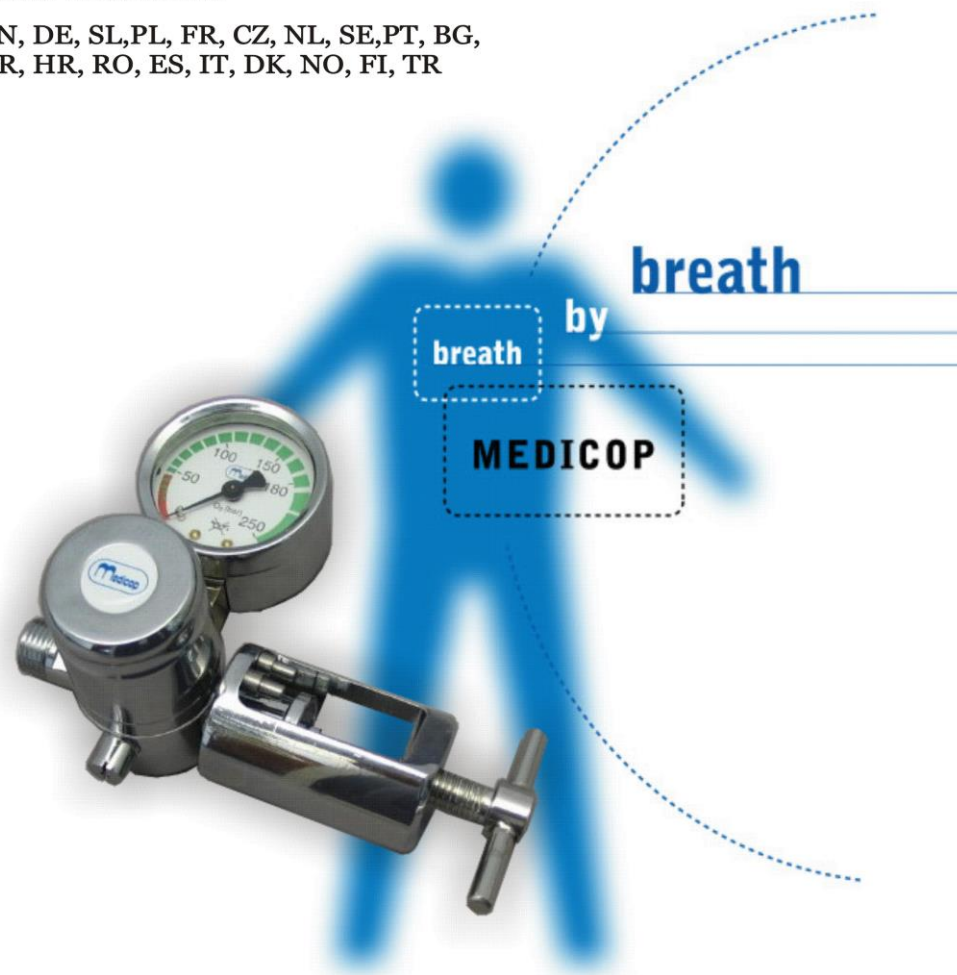EN, DE, SL, PL, FR, CZ, NL, SE, PT, BG,
GR, HR, RO, ES, IT, DK, NO, FI, TR

Medicop[®] / MEDICAL EQUIPMENT / MURSKA SOBOTA / SLOVENIA
EQUIPMENT SPECIALIST

EN	- USER´S MANUAL” PRESSURE REGULATOR FOR MEDICAL GASES”.....	4
DE	- GEBRAUCHSANWEISUNGEN “DRUCKMINDERER FÜR MEDIZINISCHE GASE“.....	6
SL	- NAVODILA ZA UPORABO “REDUCIRNI VENTIL ZA MEDICINSKE PLINE“.....	8
PL	- INSTRUKCJA OBSŁUGI “ZAWÓR REDUKCYJNY DO GAZÓW MEDYCZNYC“.....	10
FR	- MODE D’EMPLOI “SOUPAPE DE RÉDUCTION POUR LES GAZ MÉDICAUX“.....	12
CZ	- NÁVOD K POUŽITÍ “REDUKČNÍ VENTIL PRO MEDICÍNSKÉ PLYNY“.....	14
NL	- GEBRUIKSAANWIJZING “REDUCEERVENTIEL VOOR MEDISCHE GASSEN“.....	16
SE	- ANVISNINGAR FÖR ANVÄNDNING “REDUCERING VENTIL FÖR MEDICINSKA GASER“.....	18
PT	- INSTRUÇÕES PARA A UTILIZAÇÃO “VÁLVULA REDUTORA PARA OS GASES MEDICINAI“.....	20
BG	- РЕДУЦИР- ВЕНТИЛ ЗА ГАЗОВЕ ЗА МЕДИЦИНСКИ ЦЕЛИ.....	22
GR	- ΟΔΗΓΙΕΣ ΧΡΗΣΕΩΣ “ ΒΑΛΒΙΔΑ ΡΥΘΜΙΣΗΣ ΑΕΡΙΩΝ ΙΑΤΡΙΚΗΣ ΧΡΗΣΗΣ.....	24
HR	- UPUTE ZA UPORABU REDUKCIONI VENTIL ZA MEDICINSKE PLINOVE...	26
RO	- INSTRUCȚIUNI DE UTILIZARE VENTIL DE REDUCERE PENTRU GAZE MEDICALE.	28
ES	- INSTRUCCIONES DE USO VÁLVULA DE REDUCCIÓN PARA GASES MEDICINALES.....	30

NOTE

NOTE

IT	-	ISTRUZIONI D'USO VALVOLA DI RIDUZIONE PER GAS MEDICINALI.....	32
DK	-	BRUGSANVISNING REDUCERINGSVENTIL TIL MEDICINSK GAS.....	34
NO	-	BRUKSANVISNING REDUKSJONSVENTIL FOR MEDISINSKE GASSER...	36
FI	-	KÄYTTÖOHJEET LÄÄKEKAASUJEN SÄÄTÖVENTTIILI.....	38
TR	-	KULLANMA TALİMATI TIBBİ GAZLAR İÇİN EMNİYET VANAS.....	40

USER'S MANUAL PRESSURE REGULATOR FOR MEDICAL GASES-1300004

Dear customer,
We would like to thank you, that you are choosing our product. It is recommended that it will be used in accordance with these instructions for use.

The product is made and tested in accordance with following European standards: DIN EN 10524-1, ISO 14971 and annex I of directive 93/42/EEC.

The product has a marking:

INSTRUCTIONS FOR USE

TECHNICAL DATA:			
Outlet flow rate (150 bar):	40 L/min		
Max. inlet pressure (P ₁):	100 bar	Outlet pressure:	3,6 – 5,5 bar
Thread of coupling nut (HP):	DIN 477, BS 341, PIN INDEX		
Temperature:		Humidity:	
- operation	-20°C to + 60°C	0 – 95 %	
- storage	-20°C to + 70°C	non-condensing	
	The oiling of the regulator parts is prohibited!		

DESCRIPTION AND APPLICATION:

Pressure regulator is made as one-stage regulator. The regulator reduces inlet pressure (10-100 bar) to the outlet pressure 3,6 – 5,5 bar. Manometer (2) shows the pressure in the cylinder. Pressure regulator could be screwed on the cylinder by the coupling nut (1) with the hands, without any tool.

By coupling of the regulator with PIN-INDEX yoke to the gas cylinder make sure that the pins of the yoke are correctly inserted in to the cylinder valve.

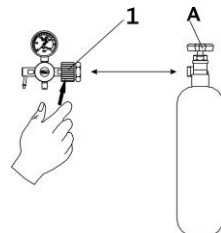
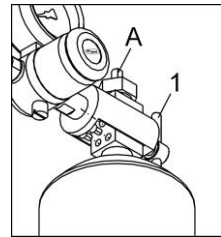
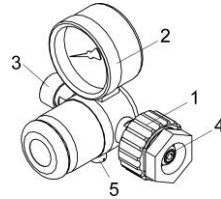
When the cylinder is emptied the cylinder has to be refilled. To perform this close the shut-off valve on the gas cylinder (A), disconnect all tubes, which are connected to the pressure regulator, empty the rest of the pressure out of regulator and unscrew the regulator by turning the coupling nut (1) to the left (with your hand only). The pressure regulator is also equipped with the safety valve (5), which in case, if the outlet pressure increase to more than 7.5 bars, lets out the gas from pressure regulator in the environment.

START OPERATING

By one slowly turn open the cylinder valve (A). Gas supply is established. Manometer indicated the pressure in the cylinder
Attention: In case of DISS outlet (3) the outlet flow is established as soon as you open the cylinder valve.

IMPORTANT NOTICE!

By use and service it is important to have suitable cleanness of the hands, workroom and tools. The pressure regulator shall be dry cleaned only.

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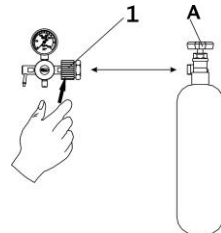
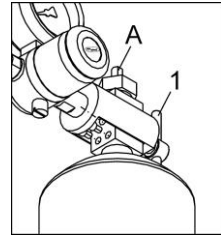
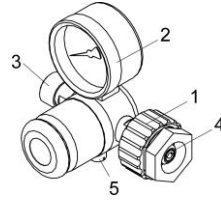
41

MAINTENANCE

PRESCRIBED REPLACEMENT OF GASKETS AND FILTERS:

Description	Ident. Nr.	Period
Sinter filter DIN connector yoke	1052082	2 years
Sinter filter BS connector yoke	1052082	2 years
Sinter filter PIN-INDEX connector yoke	1052064	2 years
Gasket DIN connector yoke; Ø10.78x2.62	1052196	2 years
Gasket BS connector yoke; Ø10.78x2.62	1052196	2 years
Gasket UNI connector yoke; Ø5.28x1.78	1053022	2 years
Gasket PIN-INDEX connector yoke; Ø16/Ø6.3	1052197	2 years
General service for DIN connector yoke	1053130	5 years
General service for PIN-INDEX connector yoke	1053131	5 years
General service for UNI connector yoke	1053132	5 years

Replacing of the gasket

1. Remove the gasket by use an appropriate dry and clean screwdriver.
2. Carefully place the new gasket in to the seat.

Replacing of the sinter filter

1. With an appropriate screwdriver unscrew the sinter filter by turning it to the left.
2. During the procedure it is recommended the pressure regulator is turned upside-down, so that dirtiness can not enter into the regulator when it is without filter.
3. Fix the new sinter filter by turning it with a screwdriver to the right.

WARNINGS!

- The oiling of the regulator parts is strongly prohibited!
- The shut-off valve of the cylinder shall be closed and opened slowly, because there is the danger of the pressure push.
- The settings of the safety valve shall not be changed!
- The shut-off valve of the cylinder shall be closed after the use!
- Don't ever let to get the cylinder completely empty but it should be refilled when there is still app. 10bar of pressure inside!

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